



Instruction Sheet Squeeze Me 148080

Store these instructions in a safe place or you may access it on our website.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location. The Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.


MATERIAL LIST

- 1 - Squeeze Me Cover
- 2 - 40cm (16" D) Peanut Ball
- 2 - Lycra Ball Cover
- 1 - Vinyl Covered Pad
- 1 - Instruction Sheet
- 1 - Maintenance Checklist

WORKING LOAD:

Setting up the Squeeze Me

1. Insert the deflated peanut ball into the lycra ball cover. Inflate ball slowly through opening and adjust cover as needed. Inflate ball to 35" L x 16"
H. Repeat with other ball.
2. Remove Lycra covering from hook fastener on the bottom of the vinyl covered pad.
Place vinyl pad inside the Squeeze Me cover and attach to loop fastener on the bottom.
3. Insert the inflated peanut balls on top of the vinyl pad.
4. Remove the Lycra covering from the hook fastener on the bottom of the Squeeze Me cover. Please store the Lycra that covers the Velcro in the cover with the balls as this will be useful when washing the cover. Attach the hook and loop together.

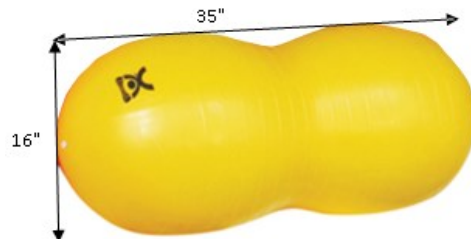

Do not use the Squeeze Me while in possession of any object that could damage the fabric or ball, such as a pencil or a comb. Your client may need assistance to enter and exit the seat.

MAINTENANCE CHECKLIST:

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

- Check for loose stitches or ripped seams. If loose stitches or ripped seams are found, discontinue use until repaired.

MATTING PRECAUTIONS:

- Always use appropriate matting.

STORAGE PROCEDURES:

- Store out of direct sunlight.
- Store where not accessible by clients without proper supervision.

CLEANING INSTRUCTIONS:

- Check for client allergies first.
- Check with your infectious disease department before using these procedures.
- Outer cover may be machine washed using cold water and mild detergent. Cover hook fastener with the Lycra covering referred to in step 4 in set up. This will prevent snagging during washing.
- Tumble dry, low heat.
- Inner vinyl cover can be cleaned with a mild disinfectant.
- Wipe ball with cloth and disinfectant.

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write Southpaw Enterprises, Inc.
P.O. Box 1047, Dayton, Ohio 45401-1047 • Toll Free - (800) 228-1698 • Please visit us at southpaw.com
© 2024 Southpaw Enterprises, Inc.