



Store these instructions along with the enclosed maintenance checklist in a safe place. You may also access them on our website.

# Instruction Sheet: Individual Bean Chair 156421



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location. The Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

**IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.**

#### MATERIALS LIST:

- |                              |                           |
|------------------------------|---------------------------|
| 1 - Bean Chair (unassembled) | 1 - Fabric Seat           |
| 2 - Side Boards              | 1 - Instruction Sheet     |
| 3 - Dowels                   | 1 - Maintenance Checklist |

**WORKING LOAD:** 150 lbs.

#### INSTRUCTIONS FOR USE:

Chair may be used on either side. One side will provide a stable base, the other will provide a rocking motion.

#### MAINTENANCE CHECKLIST:

All of the following wear points should be checked on a monthly basis and recorded on the maintenance checklist enclosed.

- Check wood for cracks or splinters. If cracked, discontinue use. For splinters, sand until smooth and refinish with clear lacquer.
- Check for loose screws. These may need to be tightened periodically, but do not over tighten.
- Check for torn cover or ripped seams. If found, discontinue use until repaired.

#### MATTING PRECAUTIONS:

Always use appropriate matting.

#### STORAGE PROCEDURES:

- Store where not accessible by clients without proper supervision.

#### CLEANING INSTRUCTIONS:

Check with your infectious disease department before using these procedures.

- Check for client allergies first.
- Wipe wood with a damp cloth using soap and water or disinfectant. Dry before using.
- Cover is machine washable - Cold water, mild detergent. Air dry - no heat.

**Safety Precautions** - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

**This sheet is for example only.** It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write  
Southpaw Enterprises, Inc.  
P.O. Box 1047, Dayton, Ohio 45401-1047 • Toll Free - (800) 228-1698  
Please visit us at [southpaw.com](http://southpaw.com)  
© 2024 Southpaw Enterprises, Inc.



**southpaw enterprises, inc.**

# EQUIPMENT MAINTENANCE CHECKLIST

Date:

Inspection Action: ( I ) Inspect ( R ) Replace

EQUIPMENT INFORMATION		WEAR POINTS															
Equipment Name:	A																
Purchase Date:	B																
Date of Installation	C																
Installation Site:	D																
Location of Instruction	E																
Sheets:	F																
NOTES:	G																
	H																
	I																
	J																
	K																
	L																
DATE	INSPECTOR	COMMENTS															

\* This sheet is for example only. It is not meant to, in any way, replace county, state, or federal standards.