



# Instruction Sheet

## VP-ASwing Platform Swing 6518

Store these instructions with the enclosed maintenance checklist in a safe place. You may also access them on our website.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

**IMPORTANT:** Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

### MATERIALS LIST:

- |                                   |                           |
|-----------------------------------|---------------------------|
| 1 - ASWING Platform Swing         |                           |
| 1 - Instruction Sheet             | 1 - Safety Snap           |
| 1 - Safety Snap Instruction Sheet | 1 - Maintenance Checklist |

**WORKING LOAD:** 300 lbs.

### INSTRUCTIONS FOR USE:

- The ASWING Platform Swing is designed for use with the Southpaw ASWING Frame. Attach the Safety Snap to the swivel eye bolt at the top of the frame.

- 

### MAINTENANCE CHECKLIST:

All of the above wear points should be checked on a monthly basis and recorded on the checklist enclosed.

- Check the eye bolt/safety snap connection point for wear. (See Safety Snap Instruction Sheet).
- Check ropes for loose or missing splice locks\*. If loose or missing splice locks are found, discontinue use until repaired.
- Check edge padding to see that it is securely attached.
- Check wood for cracks or splinters. If cracks or splinters are found, discontinue use until repaired.

\* A splice lock is the metal clip near the end of the ropes.

### STORAGE PROCEDURES:

- Store out of direct sunlight.
- Store where not accessible by clients without proper supervision.

### CLEANING INSTRUCTIONS:

- Check with your infectious disease department before using these procedures.
- Check for client allergies first.
- Dust or wipe with a damp cloth.
- Vinyl pad may be cleaned with prescribed disinfectant. Dry before using.

## **CAUTION**

- ◆ **Check eye bolt for wear and movement.**
- ◆ **Floor mats of proper thickness should be positioned properly under and around all suspended equipment.**

**Safety Precautions** - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

**This sheet is for example only.** It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

If you have any questions or comments, please call or write  
Southpaw Enterprises, Inc.  
P.O. Box 1047, Dayton, Ohio 45401 -1047  
Toll Free - (800) 228-1698  
Please visit us @ [www.southpaw.com](http://www.southpaw.com)  
© 2020 Southpaw Enterprises, Inc.